


Safe IPen

The Revolutionary Alternative

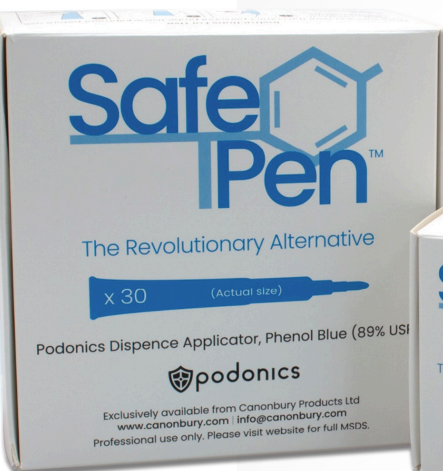

What Is A SafeTPen™?

SafeTPen™ is designed and manufactured by Podonics as a safer way to deliver phenol to the nail matrix for ablation during nail surgery procedures.

Each ampoule contains 0.4mls of phenol liquid (89% USP concentration) safely contained within a glass ampoule and has an outer plastic delivery device. The tip is made from compressed polyester. The polyethene applicator tip is tapered at the point of application to allow more accurate delivery than the cotton tips currently available on the market.


The phenol within the glass ampoule of SafeTPen™ is released by breaking the glass ampoule at the proximal and then distal end and by a gentle squeeze to allow the phenol to impregnate the tip.




Instructions For Use




Do NOT attempt to break glass ampoule, other than at marked ends.
Do NOT attempt to break in the middle or remove glass from plastic outer tube.
Make sure you wear the correct personal protective equipment before use.

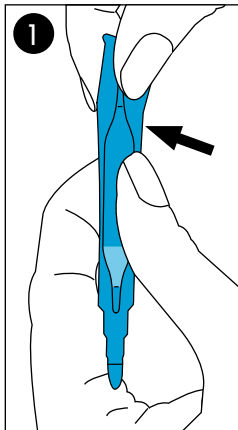

1. Snap
Using both hands to hold the SafeTPen, gently snap the top ampoule tip.

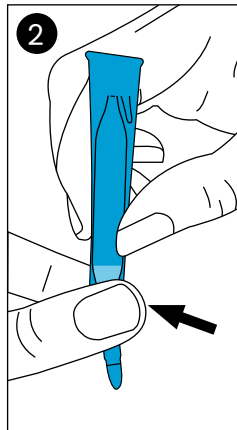

2. Snap
Repeat the same process with the bottom ampoule tip.

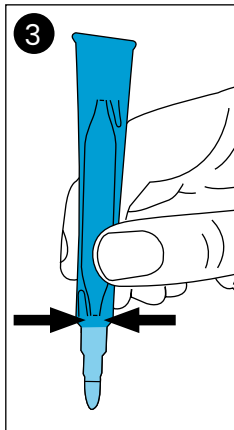

3. Squeeze
Using your fingertips, lightly squeeze the device to load the SafeTPen.

The delivery device and application allows accurate delivery of the phenol to the nail matrix where the phenol will create a chemical cauterisation during ablation. The glass ampoule and plastic delivery system significantly reduces the potential and risk of phenol coming into contact with skin, clothing or surfaces prior to the procedure.

Furthermore, by using SafeTPen™ during the ablation of the nail matrix, it prevents the risk of phenol spilling over into the sulci during delivery or over the eponychium and underneath the tourniquet.


Advantages Of Using SafeTPen™?

- Easy to use.
- Reduces possibility of a phenol spill.
- Controlled amount delivered to each patient.
- A tapered tip to assist with more accurate delivery of phenol to the nail matrix.
- Readily visible phenol through incorporation of medical dye.




0.4mls of Liquid Phenol 89% USP

Each SafeTPen™ device contains 0.4ml of Liquid Phenol 89% USP.

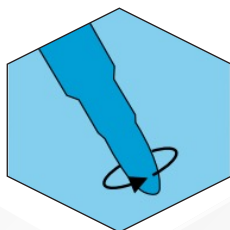


Precise Application

Ultra fine tip provides accurate and controlled delivery of Liquid Phenol to the nail matrix eliminating over use.

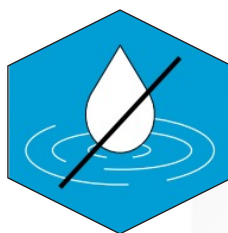


Safe, Non Splash Application

Designed for safety, all Liquid Phenol is contained within a glass ampoule and sealed inside a plastic delivery case, reducing evaporation and eliminating splash.

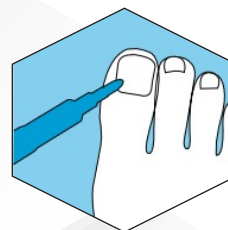

Accuracy of Delivery

SafeTPen™ Liquid Phenol has been coloured with a blue medical dye, improving the accuracy of delivery.

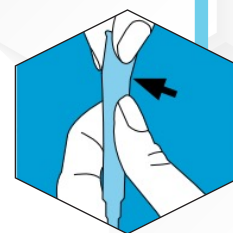

Snap, Snap, Squeeze

SafeTPen™ devices feature a lightweight and ergonomic design, activated through a simple Snap, Snap, Squeeze action.

SafeTPen™ Pack Size

SafeTPen™ is available in packs of 12 or 30





+44 7760 274874
mark@podonics.com
podonics.com

Offices:
Manchester - UK
Duerrwangen - Germany